



Shower wheelchair in aluminum h8914

Shower chair with anti-corrosion treated white aluminum frame and 4-wheel. While soft seat width 45 cm. Capacity 130 kg

Wheelchair shower ideal for showering sitting down for all the people whose mobility is extremely low. The anti-corrosive aluminum frame provides durability and the seat, backrest and footrest poliuretano soft prevent the occurrence of pressure sores.

It is equipped with 4-wheel brakes 5 "and is equipped with toilet bucket with lid in such a way that it can be used as a shower chair that as a toilet. Footrests are adjustable and removable and the seat height is adjustable

Technical detail:

- aluminum frame treated white
- 4 braked wheels 5 "
- seat, back and foot rest in soft polyurethane
- removable and adjustable footrest
- seat width 46 cm
- seat depth 43 cm
- seat height 49 to 54 cm adjustable
- max. ability (WC) 42 - 47 cm adjustable
- Overall width (with arms) 56 cm
- total depth of 93 cm
- total height from 98 to 103 cm adjustable
- maximum capacity 130 kg
- weight 12.90 kg
- CE mark (in accordance with Directive 93/42/EEC of 14 June 1993, Appendix I, concerning medical devices)
- 2 years warranty

Reversal of the arms

- this function can facilitate the ascent or descent of the wheelchair user
- in the shower, the reversal of the arms also allows a better igienecorporea during use, never leave unattended wheelchair with arms folded
- for flip arms folded armrests backward, lifting them in the front
- again to reposition the armrests, ribaltateli forward by pressing on the front

Adjusting the footrest to the desired height

- the height of the footrest of the wheelchair should be adapted to the user
- the height is adjusted properly, if at user sitting, her thighs rest properly on the seat and his feet resting on the foot rests
- to adjust the height of the footrest, remove them from the wheelchair
- remove the screw adjust the height of the footrest, inserting or removing the tube internore
- insert and tighten the screw

Adjustment of the seat

- the seat height should be adjusted to the user and is adjusted properly, if the user rests regularly sitting down to earth
- adjust the height of the seat only when the wheelchair is empty
- for height adjustment grab the clip and pull it out
- adjust the telescopic tube the height desideratare
- insert the clip into the hole provided
- always set all the telescopic tubes at the same height
- make adjustments when using the wheelchair!

INFORMATION

- **Height** adjustable
- **Width** 56 cm
- **Depth** 93 cm
- **capacity** 130 kg
- **Material** aluminum